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FIRST-VOID URINE COLLECTOR

EN: User instructions
Read all instructions prior to collection

I. Collector tube

- A. Tube cap, B. Collector tube
II. Colli-Pee® device
C. Housing, D. Floater

CAUTION

Do not wash your intimate zone thoroughly before urinating.
Do not urinate 1 hour prior use
Do not use if the packaging is damaged,
Tube cap (A) should be screwed on tightly.
Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.

1. Unscrew the tube cap (A).
2. Screw the collector tube (B) on the Colli-Pee® device.
3. Collect urine: The device will automatically collect the correct volume.
There is no need to interrupt the urine stream.
4. Disconnect the collector tube from the Colli-Pee®.
5. Close the tube tightly. You will feel and hear a "click" upon correct closure.
6. Dispose of the Colli-Pee®.
7. Wash your hands.

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In vitro diagnostic medical device
Dispositif médical de diagnostic in vitro

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Do not use if package is damaged
Ne pas utiliser si l'emballage est endommagé

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Storage instructions
Instructions de stockage

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REF N00055

Version 2019-06A

novosanis nv
medical device company

Bijkhoevelaan 32c
B-2110 Wijnegem, Belgium
welcome@novosanis.com
Subsidiary of OraSure Technologies Inc.

COLLI-PEE is a registered trademark
in the name of Novosanis NV.
www.novosanis.com

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
Milan, 17th June 2019

Subject: REVISION CERTIFICATE

We hereby certify, to the best of our ability and knowledge, that the document:

“IFU translation part”

is a true and correct translation of the English original and was thoroughly reviewed for consistency and clarity to the best of their ability and expertise by professional mother-tongue translators.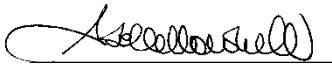

Adele Nardulli
Managing Partner