
Colli-Pee® FV-5004


REF N00290

COLLI-PEE is a registered trademark  
in the name of Novosanis NV  
[www.novosanis.com](http://www.novosanis.com)

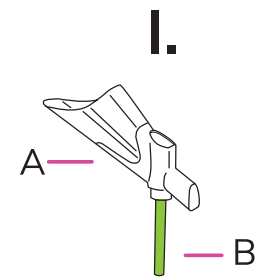II.




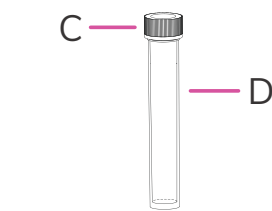
Patent ([www.novosanis.com/legal-notice](http://www.novosanis.com/legal-notice))  
© 2022 Novosanis, a subsidiary of OraSure  
Technologies, Inc., all rights reserved.

UK Responsible Person:  
**Emergo Consulting (UK) Limited**  
c/o Cr360 - UL International, Compass House, Vision  
Park Histon, Cambridge, CB24 9BZ, United Kingdom.  
[UKRPvigilance@ul.com](mailto:UKRPvigilance@ul.com)  
Tel: +44 (0) 1223.772.671

DNA Genetek,  
3000 – 500 Palladium Drive,  
Ottawa, Canada K2V 1C2

Bijkhoevelaan 32c  
B-2110 Wijnegem, Belgium  
[welc@novosanis.com](mailto:welc@novosanis.com)  
Subsidiary of OraSure Technologies, Inc.

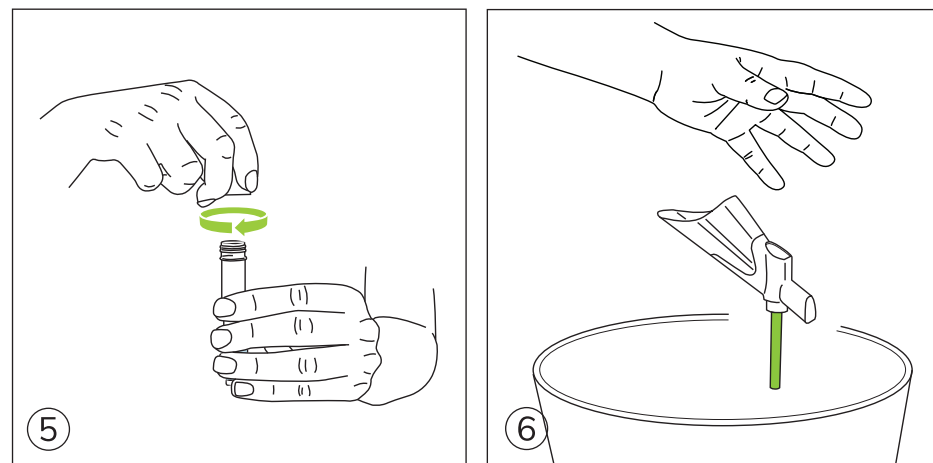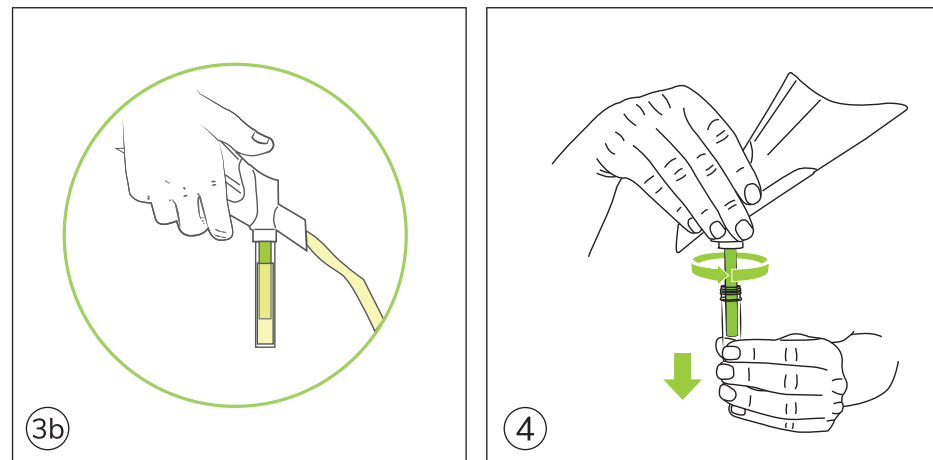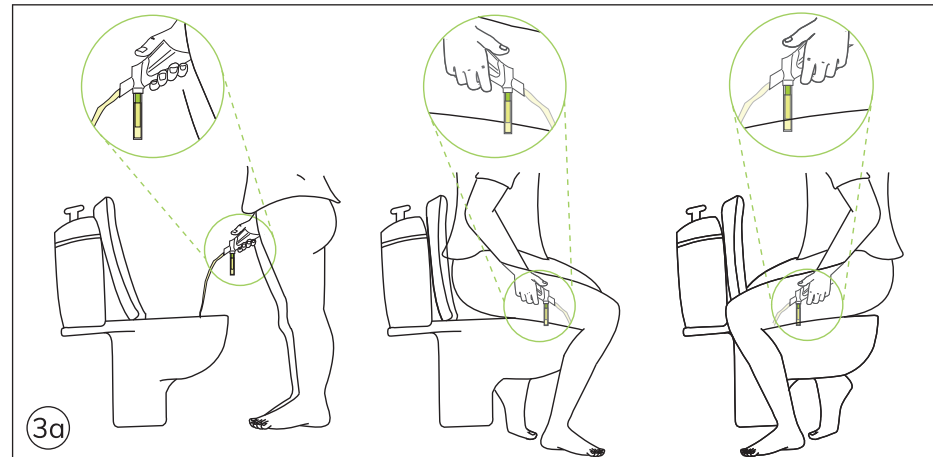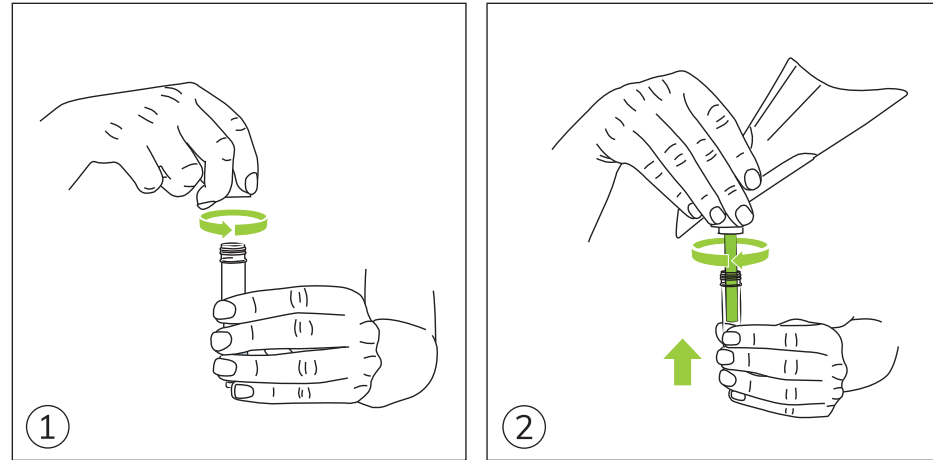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## EN FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection of first-void urine samples.  
Read all instructions prior to collection.

### ONE PACKAGE CONTAINS

- I. Colli-Pee® funnel**  
A. Housing, B. Floater  
**II. Collector tube**  
C. Tube cap, D. Collector tube

### CAUTION

Do NOT wash your intimate zone thoroughly before urinating.  
Do NOT urinate 1 hour prior to use.  
Do NOT use if the packaging is damaged.  
For single use only.

### User instructions

- Unscrew the tube cap (C).
- Screw the collector tube (D) on the Colli-Pee® funnel.
- Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.\*
- Disconnect the collector tube from the Colli-Pee® funnel.
- Close the tube tightly. Tube cap (C) should be screwed on tightly.
- Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
- Wash your hands.

\*In case of any serious incident with the device, please inform  
Novosanis and FAMHP: <https://www.famhp.be/en>

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