
Colli-Pee® FV-5010a-IUO


Investigation Use Only

REF N00237_IUO

I.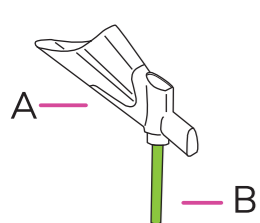





II.

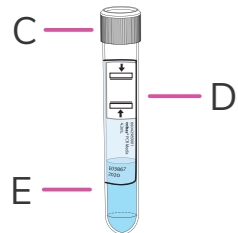COLLI-PEE is a registered trademark
in the name of Novosanis NV
www.novosanis.com

Patent (www.novosanis.com/legal-notices)
© 2022 Novosanis, a subsidiary of OraSure
Technologies, Inc., all rights reserved.

novosanis nv
medical device company
Bijkhoevelaan 32c
B-2110 Wijnegem, Belgium
welcome@novosanis.com
Subsidiary of OraSure Technologies, Inc.

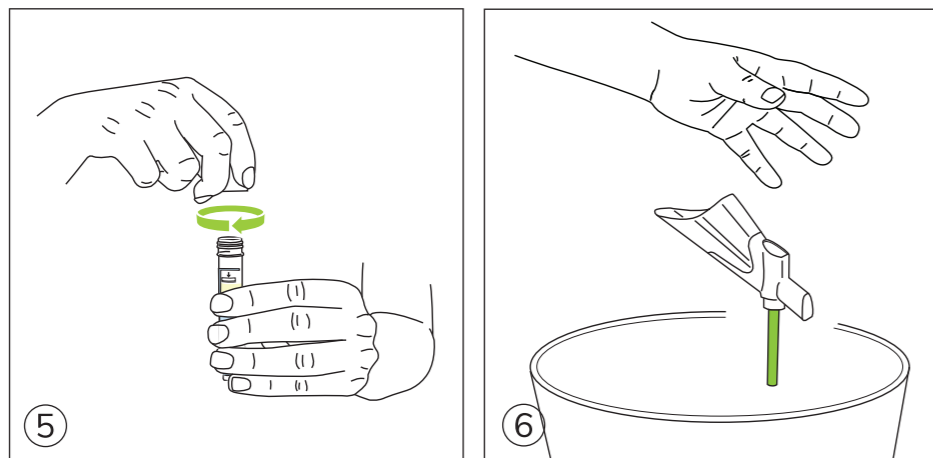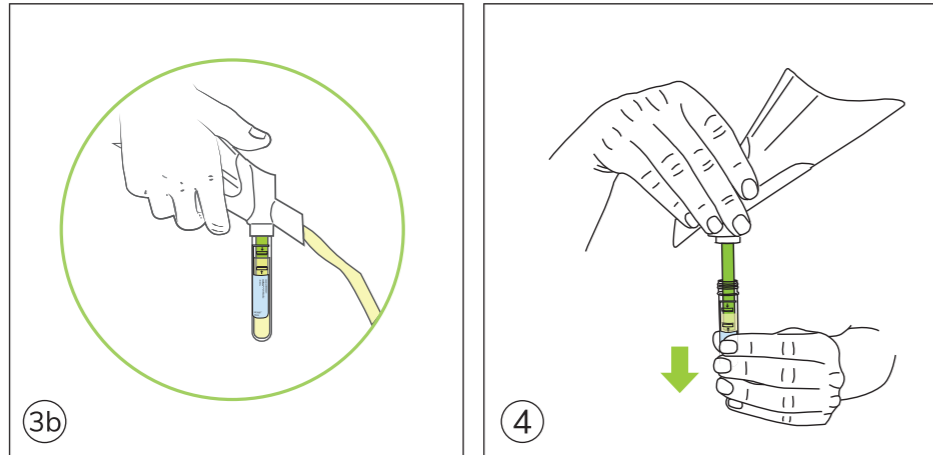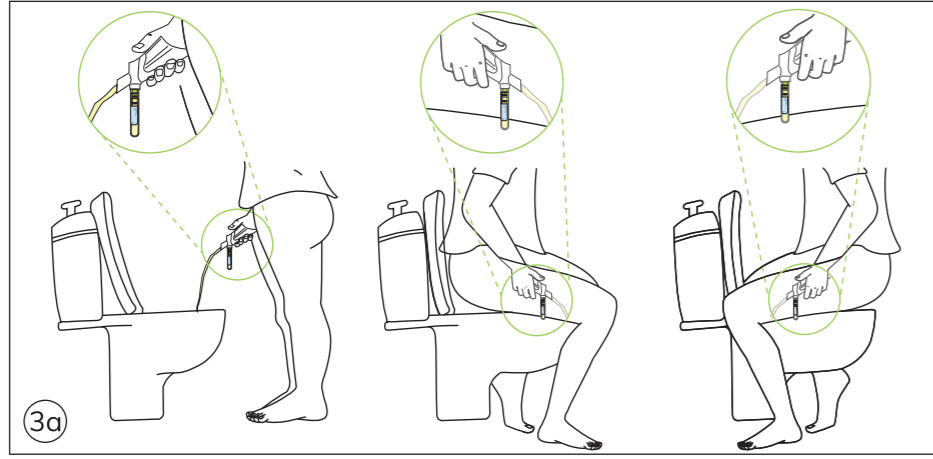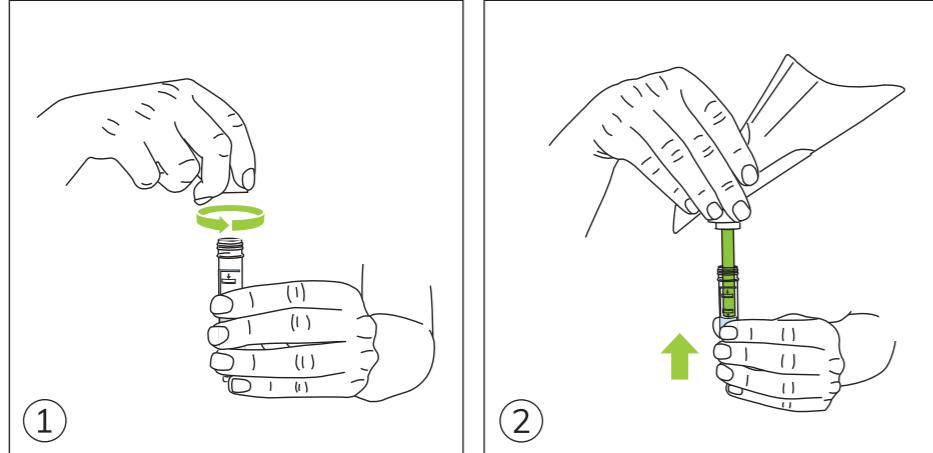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EN
FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection and preservation of first-void urine samples.
Read all instructions prior to collection

ONE PACKAGE CONTAINS**I. Colli-Pee® funnel**

A. Housing, B. Floater

II. Collector tube

C. Tube cap, D. Collector tube, E. Preservative 'Cobas PCR media'

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.
Preservative should remain inside collector tube. Do not drink the preservative.*

User instructions

1. Unscrew the tube cap (C).
2. Push the collector tube (D) on the Colli-Pee® funnel.
3. Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.**
4. Disconnect the collector tube from the Colli-Pee®.
5. Close the tube tightly. Tube cap (C) should be screwed on tightly.
6. Dispose the Colli-Pee® according to local regulations for plastic waste.
7. Wash your hands.

*More information can be found on the SDS, www.novosanis.com
**In case of any serious incident with the device, please inform
Novosanis and FAMHP: <https://www.famhp.be/en>

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