
Colli-Pee® FV-5020c

REF N00238

COLLI-PEE is a registered trademark in the name of Novosanis NV www.novosanis.com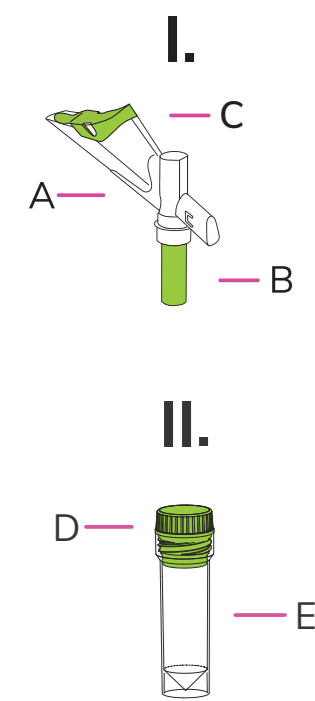






Patent (www.novosanis.com/legal-notice)
© 2022 Novosanis, a subsidiary of OraSure Technologies, Inc., all rights reserved.

UK Responsible Person:
Emergo Consulting (UK) Limited
c/o Cr360 - UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ, United Kingdom.
UKRPvigilance@ul.com
Tel: +44 (0) 1223.772.671

DNA Genetek,
3000 – 500 Palladium Drive,
Ottawa, Canada K2V 1C2

Bijkhoevelaan 32c
B-2110 Wijnegem, Belgium
welc@novosanis.com
Subsidiary of OraSure Technologies, Inc.

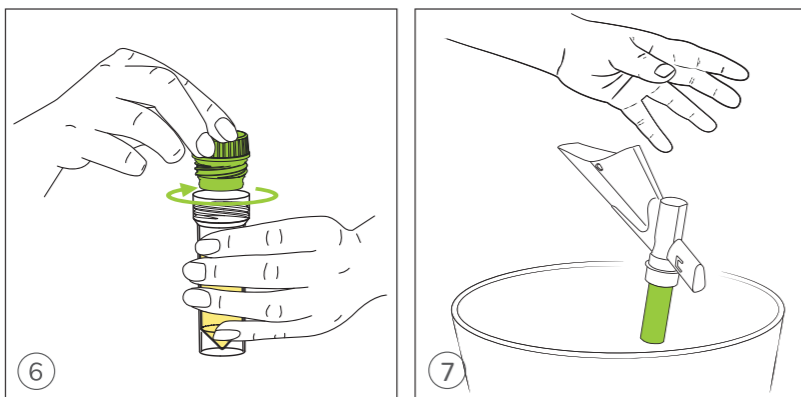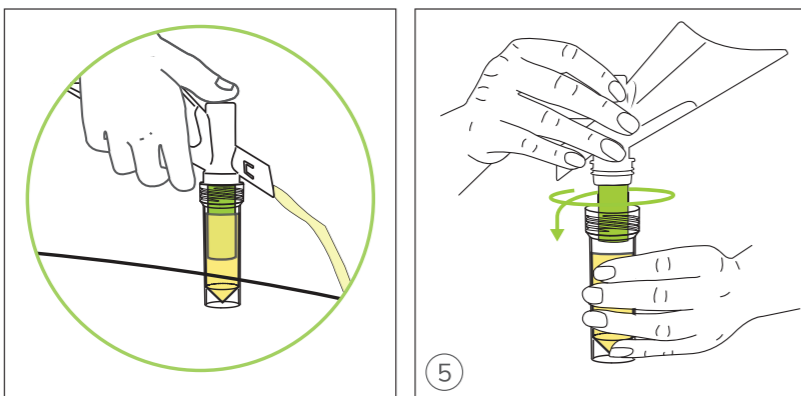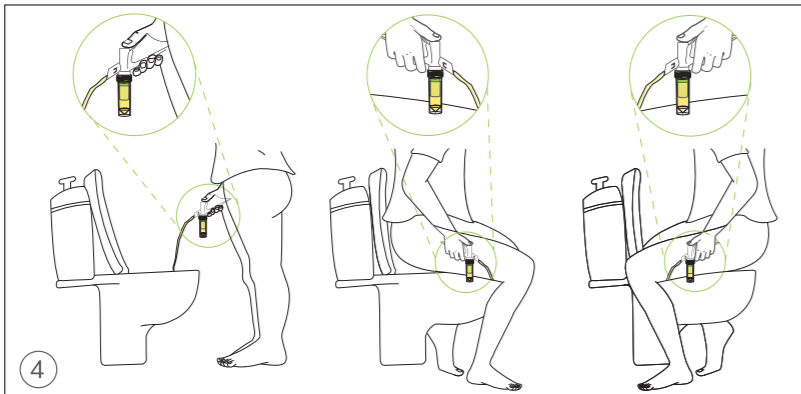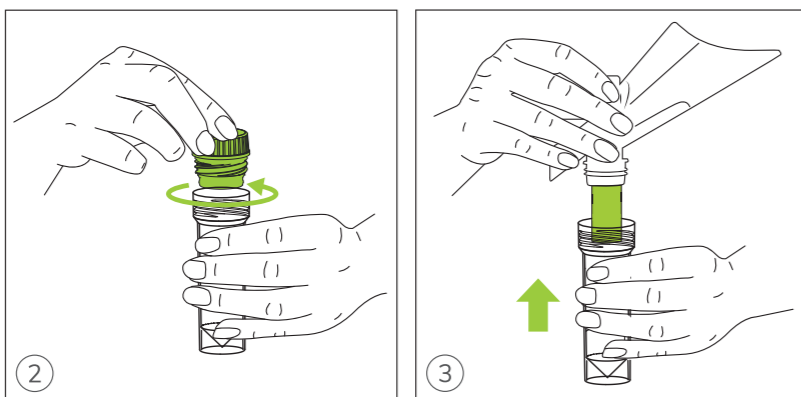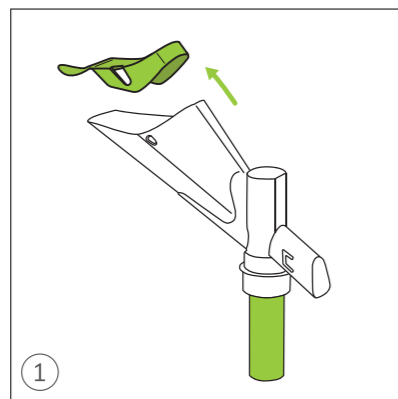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EN FIRST-VOID URINE SELF-SAMPLING DEVICE

For standardized, volumetric collection of first-void urine samples.
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

- I. Colli-Pee® funnel**
A. Housing, B. Floater, C. Transportation clip
II. Collector tube
D. Tube cap, E. Collector tube

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.

User instructions

1. Remove the transportation clip (C) from the housing (A).
2. Unscrew the tube cap (D).
3. Screw the collector tube (E) on the Colli-Pee® funnel.
4. Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.*
5. Disconnect the collector tube from the Colli-Pee® funnel.
6. Close the tube tightly. Tube cap (D) should be screwed on tightly.
7. Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
8. Wash your hands.

*In case of any serious incident with the device, please inform Novosanis and FAMHP: <https://www.famhp.be/en>

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