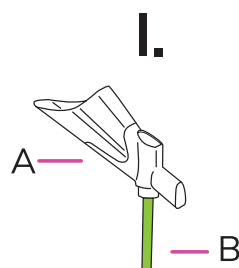
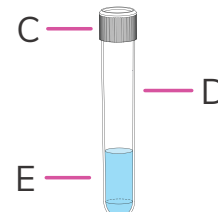
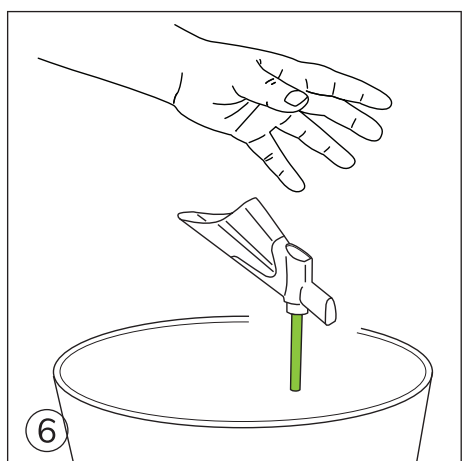
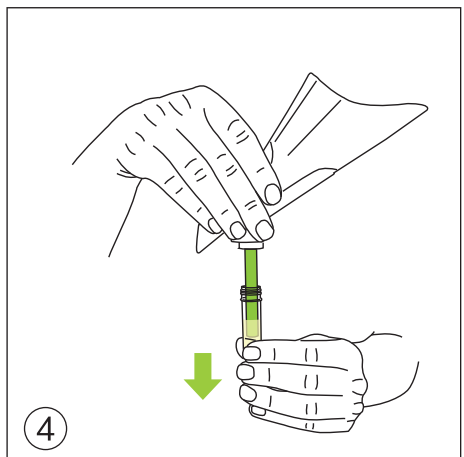
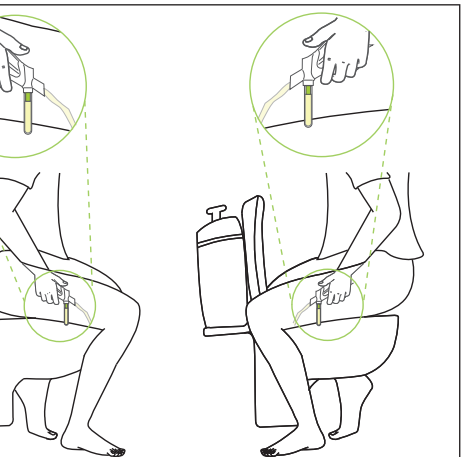
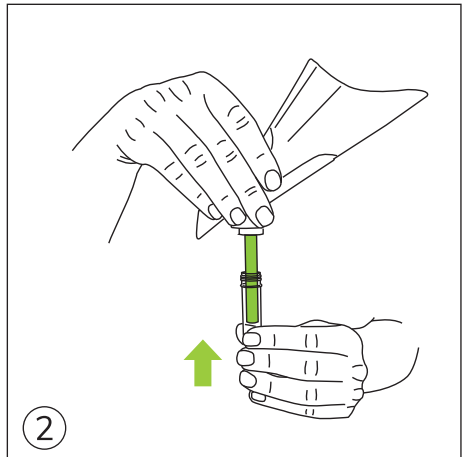
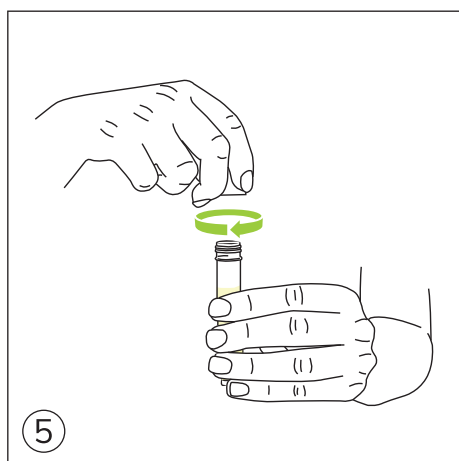
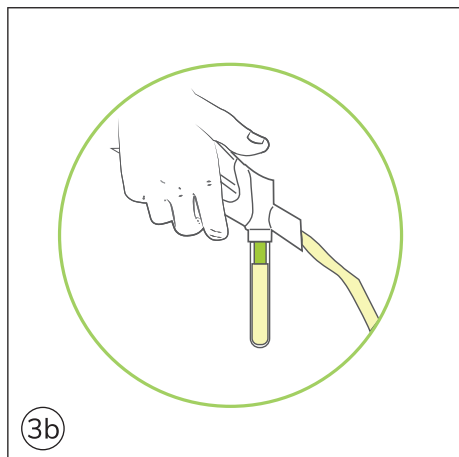
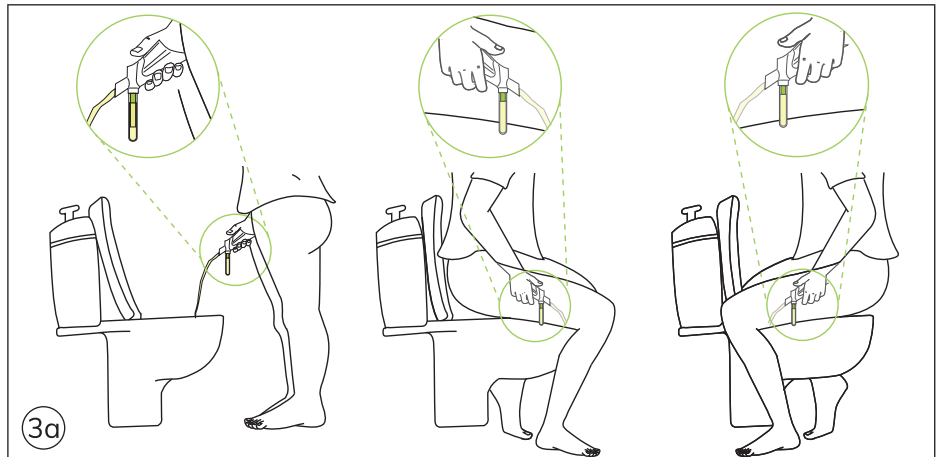
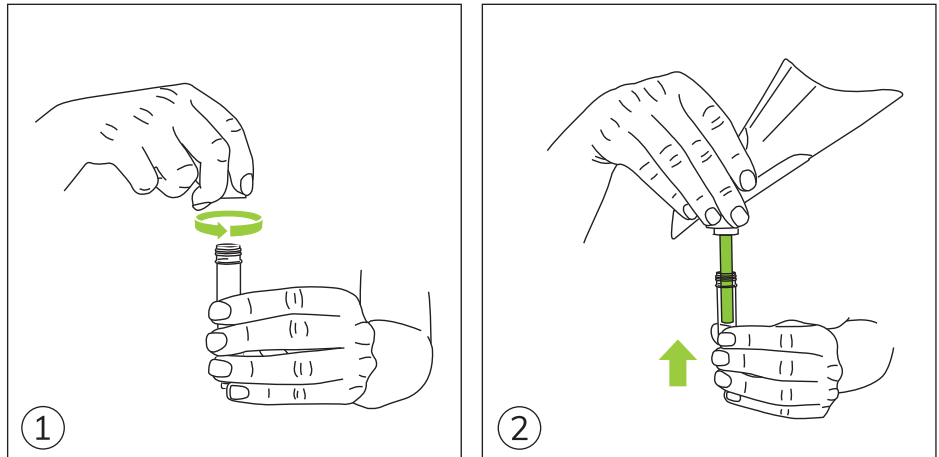
Colli-Pee® UCM® FV-5010

REF N00327

COLLI-PEE is a registered trademark
in the name of Novosanis NV
www.novosanis.com

II.

Patent (www.novosanis.com/legal-notices)
© 2022 Novosanis, a subsidiary of OraSure
Technologies, Inc., all rights reserved.UK Responsible Person:
Emergo Consulting (UK) Limited
c/o C1360 - UL International, Compass House, Vision
Park Histon, Cambridge, CB24 9BZ, United Kingdom.
UKRPvigilance@ul.com
Tel: +44 (0) 1223.772.671 DNA Genetek,
3000 – 500 Palladium Drive,
Ottawa, Canada K2V 1C2 **novosanis** nvBijkhoevelaan 32c
B-2110 Wijnegem, Belgium
welcoming@novosanis.com
Subsidiary of OraSure Technologies, Inc.**NL
EERSTE FRACTIE URINE ZELFCOLLECTOR**

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**EN
FIRST-VOID URINE SELF-SAMPLING DEVICE**

For standardized, volumetric collection and preservation of first-void urine samples.
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

- I. Colli-Pee® funnel**
A. Housing, B. Floater
II. Collector tube
C. Tube cap, D. Collector tube, E. Preservative 3.4 mL UCM®

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.

For single use only.
Preservative should remain inside collector tube. Do not drink the preservative. Wash with water if preservative comes into contact with skin or eyes.*

User instructions

- Unscrew the tube cap (C).
- Push the collector tube (D) on the Colli-Pee® funnel.
- Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.**
- Disconnect the collector tube from the Colli-Pee® funnel.
- Close the tube tightly. Tube cap (C) should be screwed on tightly.
- Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
- Wash your hands.

*More information can be found on the SDS, www.novosanis.com

**In case of any serious incident with the device, please inform
Novosanis and FAMHP: <https://www.famhp.be/en>

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