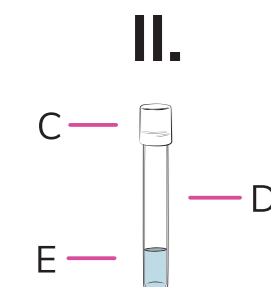
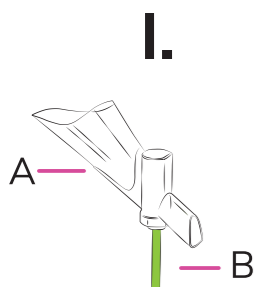
Small Volumes containing UCM

REF N00327 CE

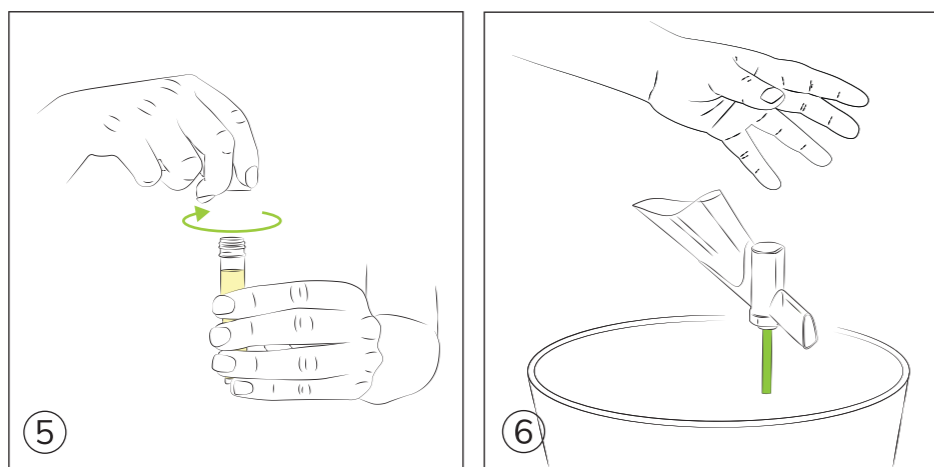
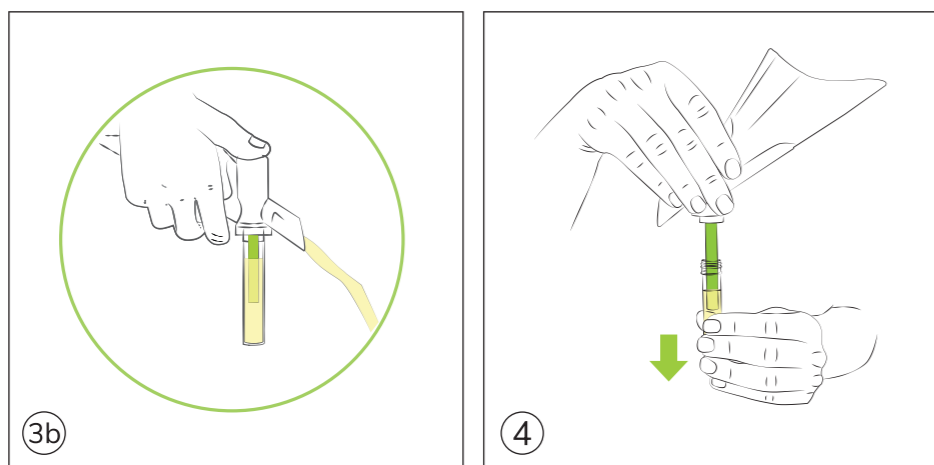
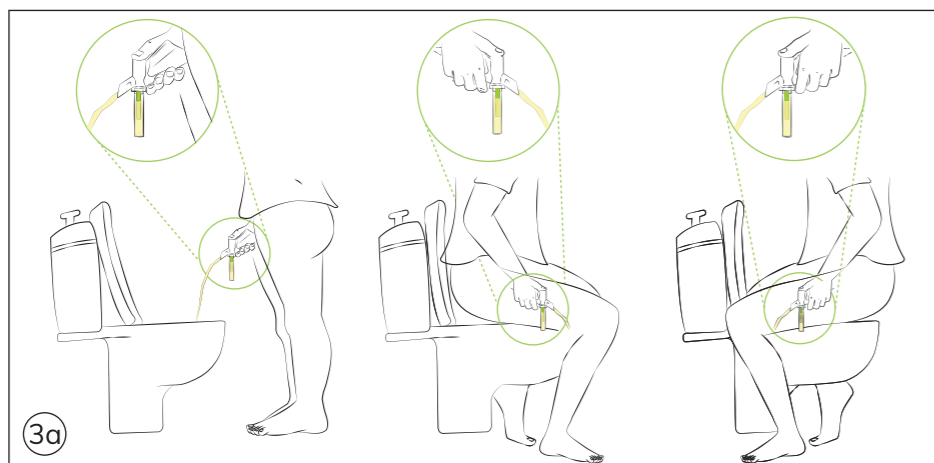
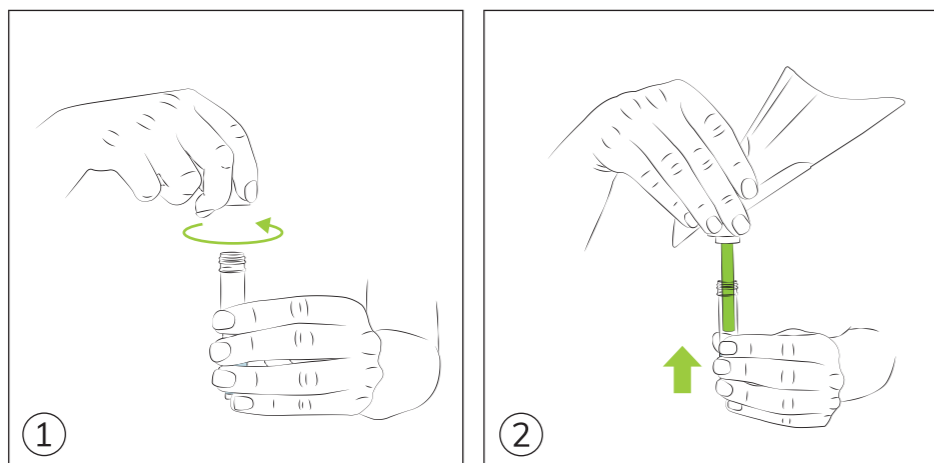
COLLI-PEE is a registered trademark in the name of Novosanis NV. www.novosanis.com

Australian Sponsor
Emergo Australia
Level 20 Tower II, Darling Park
201 Sussex Street, Sydney, NSW 2000
Australia

novosanis nv
Bijkhoevelaan 32c
B-2110 Wijnegem, Belgium
welcome@novosanis.com
Subsidiary of OraSure Technologies, Inc.

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FIRST-VOID URINE SELF-SAMPLING DEVICE
For standardized, volumetric collection and preservation of first-void urine samples.
EN: User instructions
Read all instructions prior to collection

I. Colli-Pee® device
A. Housing, B. Floater
II. Collector tube
C. Tube cap, D. Collector tube, E. Preservative

CAUTION
Do NOT wash your intimate zone thoroughly before urinating.
Do NOT urinate 1 hour prior to use.
Do NOT use if the packaging is damaged.
For single use only.
Preservative should remain inside collector tube. Do not drink the preservative.

1. Unscrew the tube cap (C).
2. Push the collector tube (D) on the Colli-Pee device.
3. Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.
4. Disconnect the collector tube from the Colli-Pee®.
5. Close the tube tightly. Tube cap (C) should be screwed on tightly.
6. Dispose of the Colli-Pee®.
7. Wash your hands.

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